
THUASNE

LYMPHATREX Essential

- (fr) Dispositif de compression sur mesure patient
- (en) Made-to-measure patient compression device
- (de) Für Patienten maßgefertigtes Kompressionsprodukt
- (nl) Compressiehulpmiddel op maat van de patiënt
- (it) Dispositivo di compressione su misura paziente
- (es) Dispositivo de compresión a medida del paciente
- (pt) Dispositivo de compressão à medida do paciente
- (da) Kompressionsprodukt efter patientens mål
- (fi) Kompressiotuote potilaan mittojen mukaan
- (sv) Individianpassad kompressionsprodukt
- (el) Διατάξη συμπίεσης στα μέτρα του ασθενούς
- (cs) Kompresivní pomůcka na míru pacienta
- (pl) Wyrób uciskowy wykonywany dla pacjenta na miarę
- (lv) Pacientam pielāgots kompresijas izstrādājums
- (lt) Pagal paciento užsakymą pagaminta kompresinė priemonė
- (et) Patsiendile eritellimusel valmistatav kohandatud kompressioontoode
- (sl) Kompresijski pripomoček po meri bolnika
- (sk) Kompresívna pomôcka vyrobená na mieru pacienta
- (hu) A páciens méretére készült kompressziós eszköz
- (bg) Персонализирано компресионно изделие за пациента
- (ro) Dispozitiv de compresie adaptat la măsura pacientului
- (ru) Компрессионные изделия по меркам пациента
- (hr) Kompresijski proizvod po mjeri bolesnika
- (zh) 为患者定制的压力矫形器
- (ja) オーダーメイドの圧迫医療器具
- (ko) 환자 맞춤형 압박스타킹
- (ar) لباس ضاغط على مقياس المريض

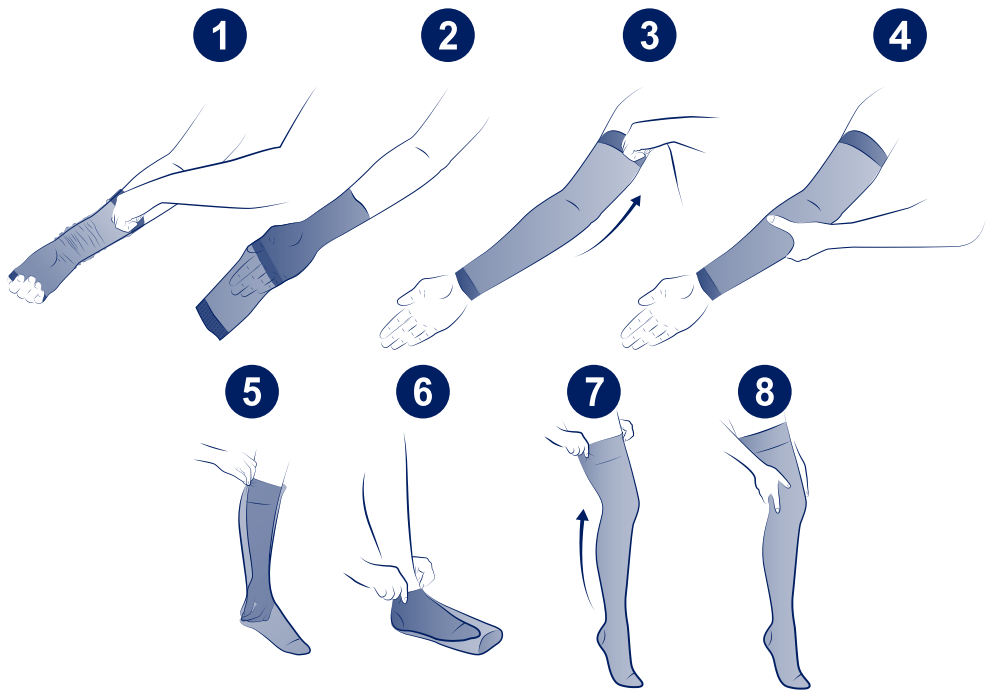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MADE-TO-MEASURE PATIENT COMPRESSION DEVICE

en

Description

LYMPHATREX Essential is a range of compression devices made to the measures of the patient's limb by circular knit. This range is available in several models:

Upper limb	Lower limb
<ul style="list-style-type: none">• Armsleeve• Armsleeve with mitten• Mitten	<ul style="list-style-type: none">• Sock• Stocking• Tights

Mechanism of action

LYMPHATREX Essential devices provide compression by applying pressure to the limb; they thus promote lymph circulation, help prevent oedema formation and maintain its volume. They are intended for use in adult or child patients suffering from at least one of the device's indications.

They are available in multiple AFNOR compression classes:

- Class 2 (15-20 mmHg)
- Class 3 (20-36 mmHg)

Indications

LYMPHATREX Essential compression devices are indicated in the treatment of primary or secondary lymphoedema of the upper or lower limbs:

- Class 2: additional treatment for the lymphoedema maintenance phase
- Class 3: during the maintenance phase, after initial reduction of the affected limb, to maintain the reduced volume.

Destination: The device is intended to be used directly on the skin of the upper or lower limbs.

Fitting instructions

Check the product is not damaged before use.

LYMPHATREX Essential devices should be worn next to the skin.

Seek advice from a healthcare professional.

Putting on the armsleeve, armsleeve with mitten or mitten

- Place your hand inside the garment and turn it back on itself to the hand opening. **1 2**
- Place your hand inside the garment, then carefully slide it over the limb to under the armpit (armsleeve and armsleeve with mitten), or to the wrist (mitten). **3**
- Avoid pulling the garment any more than necessary.
- Adjust the garment to ensure there are no wrinkles along the length of the limb.
- Once the garment is properly in place, massage the limb to ensure uniform compression. **4**

Putting on socks, stockings and tights

- Place your hand inside the garment and turn it back on itself to the heel. **5**
- Slip your foot into the garment until the heel is correctly positioned. **6**
- Carefully slide the garment over the limb to below the knee (sock), below the gluteal fold (stocking), or to the hip (tights). **7**
- Avoid pulling the garment any more than necessary.
- Adjust the garment to ensure there are no wrinkles along the length of the limb.
- Stocking or sock with anti-slip system: when putting on the garment, be careful not to grip the anti-slip part to avoid any damage.
- Once the garment is properly in place, massage the limb to ensure uniform compression. **8**

Contraindications

Do not use LYMPHATREX Essential compression devices in the event of:

- Known allergies to the materials used
- Weeping or major dermatological conditions of a limb concerned
- Phlegmasia cerulea dolens (painful blue phlebitis with arterial compression) of a limb concerned
- Septic thrombosis of a limb concerned

Do not use LYMPHATREX Essential lower limb compression devices in the event of:

- Advanced diabetic microangiopathy (for compression > 30 mmHg)
- Heart failure
- Peripheral artery disease (PAD) with ankle-brachial pressure index (ABPI) < 0.6

The risk/benefit ratio must be regularly re-assessed by a healthcare professional in the event of:

- PAD with ABPI between 0.6 and 0.9
- Advanced peripheral neuropathy
- Weeping or eczematous dermatitis

Precautions

- Take care not to damage the device when fitting it, particularly with your nails.
- LYMPHATREX Essential compression devices are designed to be worn during the day in order to maintain the volume reduction achieved by intensive treatment.
- These devices are made to the patient's measures to exert the necessary pressure recommended by the healthcare professional. They should not be used by another person as this could present risks.
- Achieving the necessary pressure may require the overlaying of several compression devices.
- Treatment success requires that the device be worn regularly.
- Do not apply the product directly to an open wound.
- Before using a compression device, ensure that any skin lesions are fully covered with an appropriate dressing.
- Ensure that the anti-slip system can be applied to healthy undamaged skin.
- Closely follow your healthcare professional's prescription and recommendations for use.
- In the event of significant discomfort, changes in limb volume or unusual sensations, remove the device and seek the advice of a healthcare professional.
- Due to the constantly evolving nature of the condition, regular monitoring by a healthcare professional is recommended to ensure that the garment is still suitable for your condition.
- Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Unwanted side effects

This device can cause skin reactions, such as redness, itching, blistering, burns or blisters.

Washing

It is recommended that the garment be washed before it is used for the first time.

Wash your garment every day, or after each use.

LYMPHATREX Essential compression devices are designed for daily use for periods of up to 6 months (i.e. 90 washes).

They should then be replaced to ensure optimum product efficiency.

Machine washing

- Machine wash at 40 °C.
- Use a washing product without softener. Do not use detergents, fabric softeners or aggressive products (products containing chlorine).
- Place the garment in a wash bag to make sure other garments (particularly those with touch fasteners or zips) do not damage the knit of your orthosis.
- Dry well away from any direct heat source (radiator, sun, etc.).
- Do not tumble-dry.
- Do not iron.

Storage

Store the device at room temperature and away from direct sunlight.

Composition

Lower limb: polyamide, elastane, polyester, polyethylene, cotton, polyurethane

Upper limb: polyamide, elastane, polyester, polyacetal, cotton, polyurethane

Anti-slip system: silicone, elastane, polyamide, polyester

Disposal

LYMPHATREX Essential compression devices should be disposed of as per local regulations.

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Thuasne Deutschland GmbH
Thuasne Benelux B.V.
Thuasne Scandinavia AB
Thuasne UK Ltd
Thuasne (Shanghai) Medical Devices Co., Ltd.